



WANDY TV Sherry International,
No.49, Lane 302, Fulin St Rd., Xi-Zhi Dist.,
Taipei, Taiwan 100

Manufactured in Taiwan for
Smartmuscle Technologies Pte Ltd,
428-01 024 Centre St, 4 Shenton Way,
Singapore 068607

PowerDot is a registered trademark of
Smartmuscle Technologies Pte Ltd.

4 SETS

8 x ROUND
4 x RECTANGULAR



0 799435 692860



FDA 21CFR801.326 (H02998) &
PMD 35-42/5EC Compliant



ELECTRODE PADS



INSTRUCTIONS

1. For best results, clean the area(s) of skin application with soap and water or alcohol wipe before use. Let dry completely.
2. Gently peel the electrode(s) off release liner(s) and place it on the skin application area. Always save release liners. When attaching electrodes, make sure the entire surface is in contact with the skin area.
3. To prevent hydrogel from drying up, keep the electrodes on the release liners when they're not in use.

These self-adhesive electrodes are an accessory for PowerDot PD-01 Connected Sports Muscle Stimulator and should only be used with this product.



WARNING!

If you develop any long-lasting skin rash or irritation from the stimulation with PowerDot electrode(s), please discontinue using these electrode(s) at once and seek medical advice.

STORAGE CONDITIONS

Store electrodes in a cool and dry place.
(Suggested Temp. <28°C; Humidity 50+/-5%), to avoid exposure of the surface to heat and humidity.

CAUTION

1. Do not use electrode(s) over broken skin or open wound(s).
2. Do not pull electrode(s) off skin during active stimulation.
3. When electrode(s) eventually loses its adhesiveness or becomes too dirty, replace it with a new electrode(s).
4. For hygienic reasons, do not use same electrode(s) on different people.

GETPOWERDOT.COM