



WARNING!

If you develop any long-lasting skin rash or irritation from the stimulation with PowerDot electrode(s), please discontinue using these electrode(s) at once and seek medical advice.

STORAGE CONDITIONS

Store electrodes in a cool and dry place (Suggested Temp. <math><28^{\circ}\text{C}</math>; Humidity 50+/-5%), to avoid exposure of the surface to heat and humidity.

CAUTION

1. Do not use electrode(s) over broken skin or open wound(s).
2. Do not pull electrode(s) off skin during active stimulation.
3. When electrode(s) eventually loses its adhesiveness or becomes too dirty, replace it with a new electrode(s).
4. For hygienic reasons, do not use same electrode(s) on different people.



WANDY T.Y. Sherry International,
No.48, Lane 392, FuDe 1st Rd., Xi-Zhi Dist.,
Taipei, Taiwan 806

**Manufactured in Taiwan for
Smartmissimo Technologies Pte Ltd,**
#28-01 SGX Centre II, 4 Shenton Way,
Singapore 068807

PowerDot is a registered trademark of
Smartmissimo Technologies Pte Ltd.




FDA 21CFR882.1320 (K132998) &
MDD 93/42/EEC Compliances



ELECTRODE PADS

These self-adhesive electrodes are an accessory for PowerDot PD-01 Connected Sports Muscle Stimulator and should only be used with this product.

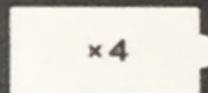

INSTRUCTIONS

1. For best results, clean the area(a) of skin application with soap and water or alcohol wipe before use. Let dry completely.
2. Gently peel the electrode(s) off release liner(s) and place it on the skin application area. Always save release liners. When attaching electrodes, make sure the entire surface is in contact with the skin area.
3. To prevent hydrogel from drying up, keep the electrodes on the release liners when they're not in use.

GETPOWERDOT.COM